

2-piece Colostomy




Stoma
Skin
Soul

1



2

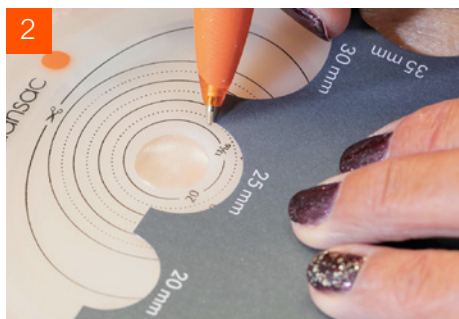

3



4



5



6



2-piece Colostomy



For further information or to order a sample, contact Customer Care on:

1800 880 851 (AUS)
dansac.com.au

0800 678 669 (NZ)
dansac.co.nz

WARNINGS

Reuse of a single use product may create potential risk to the user such as poor adhesion. Reprocessing, cleaning, disinfection, and sterilisation may compromise the product. This may result in odour or leakage. Ensure proper fit as instructed. If the barrier opening is incorrect or leakage occurs, skin damage may occur (e.g., irritation, broken skin, rash, or redness). This may decrease wear time.

Notes:

To help ensure proper adhesion, apply to clean, dry intact skin. Do not use oils or lotions. These products may interfere with adhesion. Product fit is affected by many factors such as stoma construction, stoma output, skin condition, and abdominal contours.

Contact a stoma care nurse or healthcare practitioner for help with the ostomy pouching system or skin irritation. In case of serious injury (incident) in relation to your use of the product, please contact your local distributor or manufacturer, and your local competent authority. For more information, see www.dansac.com/authority or local distributor.

Prior to use, be sure to read the Instructions for Use for information regarding Intended Use, Contraindications, Warnings, Precautions, and Instructions.